## Slovensky

### PODLOŽNÁ MISA S VRCHNÁKOM • 50076

#### POPIS

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### UNDERLYING BED-PAN WITH COVER • 50076

#### DESCRIPTION

Polypropylene bed-pan with a handle and a cover.

#### USE

The bed-pan is used to hold up stool of immobile patients confined to bed.

#### INSTRUCTION FOR USE

1. Before the first use, unwrap the bed-pan and wash it with tepid water and a usual (liquid) detergent for washing the dishes. The bed-pan may be washed manually or by a washing device determined for this purpose. Mop the bed-pan after washing.
2. Uncover the bed-pan and put it under the lying patient.
3. After the patient has relieved himself/herself, take the bed-pan away and cover it.
4. Throw out the content of the bed-pan to a container determined for this purpose (WC).
5. Wash the bed-pan with a usual liquid detergent (not with a sand). In case of need (infectious disease of the patient) disinfect the bed-pan.
- 5a. Washing device: if you use it, proceed according to the manual for the device.
6. Mop the bed-pan or dry it.
7. Put the bed-pan to the determined place (cupboard, shelf).

#### CAUTION

1. Do not expose the bed-pan to fire, radiant heat, chemical vapours, UV radiation.
2. The bed-pan is not for playing purposes, keep it away from the reach of children.
3. Use the bed-pan only for the determined purposes, (see par. USE).
4. Return a damaged bed-pan to your distributor.

#### TERMS OF GUARANTEE

1. The guarantee period is 24 months.
2. The guarantee only applies to evident defects caused by the manufacturer.
3. The guarantee does not apply to mechanical, chemical and other damages caused by negligence or failure to observe this manual.

#### ACKNOWLEDGEMENT

Thank you for purchasing this product. We believe that it will meet your needs and help you to overcome your health problems and facilitate the care of lying patients by the health staff.

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